

How to use this booklet



- 1) Treat where and how your prescriber instructed you to treat.
- 2) If no instructions were provided by your prescriber for how to treat, the following pictorials are options; you may call your Regenesi s Clinical Nurse Coordinator at 877.970.4970.
- 3) Always aim the white side with the STARBURST towards the treatment area.

Quick Start Guide



Step 1: Plug in the power cord. The screen will display the Provant Infinity logo and the light will flash.



In about 30 seconds, you will hear a beep, the light will turn green, and you will see the START screen.



Step 2: Place the white STARBURST side of the treatment pad against your treatment area.



Step 3: Press START. The device will count down the 30-minute treatment. Once treatment is complete, you will hear a beep and the device will enter sleep mode.



If the device remains plugged in, to start a new treatment press the POWER button and follow these steps again.